



**CORFLEX**

**CORFLEX IS A GLOBALLY BASED  
MANUFACTURER AND DISTRIBUTOR OF QUALITY  
ORTHOPEDIC AND REHABILITATION PRODUCTS**



**HINGED KNEE SUPPORTS / NEOPRENE**

**Contender Knee Brace**

- > Features R.O.M. hinges with contoured condyle shells that allow for full flexion and extension control along with convenience of a sleeve below the knee and benefits of an adjustable wrap above the knee; available in two lengths
- > Indicated for medial/lateral instability, muscle/ligament weakness or injury, ACL/PCL moderate instability, strains or sprains, and degenerative joint disease where extension/flexion control is required
- > Manufactured from premium grade latex-free neoprene with suede hinge cover



**PRODUCT NUMBER**

3/16"	S	M	L	XL	2XL	3XL	4XL
13"	88-8403	88-8404	88-8405	88-8406	88-8407	88-8408	88-8409
16"	88-8453	88-8454	88-8455	88-8456	88-8457	88-8458	88-8459

**SIZING GUIDE**

	6" ABOVE MID-PATELLA	MID-PATELLA	6" BELOW MID-PATELLA
Small	16"-18"	12"-14"	11"-12"
Medium	18"-19"	14"-16"	13"-14"
Large	19"-22"	16"-18"	15"-16"
X-Large	22"-24"	18"-20"	17"-18"
2X-Large	24"-26"	20"-22"	19"-20"
3X-Large	26"-28"	22"-24"	21"-22"
4X-Large	28"-30"	24"-26"	23"-24"

**APPLICATION INSTRUCTIONS**

1. Open R.O.M. hinge cover, set hinge by placing pins in desired setting and close hinge cover. Make sure both hinges have identical settings.
2. Loosen straps and open posterior closure portion of the brace.
3. Slide foot through the sleeve portion of the brace and place so patella opening is over kneecap. Make sure center axis of hinge is aligned with the center of the knee joint.
4. Secure top portion behind the knee to desired comfort level.
5. Starting with strap nearest foot, secure strap through buckle, tighten to desired comfort level. Attach remaining strap in the same manner.

**CARE & CLEANING**

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Remove any metal stays if possible, close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

**HOW TO PLACE AN ORDER**

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



**TELEPHONE**

1-800-426-7353, Ext1



**FAX**

1-603-623-4111



**E-MAIL**

Service@corflex.com



**MAIL**

CORFLEX, Inc.  
Attn: Customer Service  
669 East Industrial Park Dr  
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109  
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.