



CORFLEX.

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HAND / FOREARM SPLINTS

TARGET MP ULNAR DRIFT SUPPORT

- > Provides support and maintains neutral alignment of the MP joints while preserving mobility of the fingers
- > Indicated for treatment of Arthritis and MP joint deformity known as ulnar drift
- > Manufactured from latex-free soft loop neoprene with contact closure



PRODUCT NUMBER

	S/M	M/L
Right	88-8500	88-8501
Left	88-8510	88-8511

SIZING GUIDE

	S/M	M/L
	2 ½"-3 ½"	3 ½"-4 ½"

Measurement of width at MP joints

APPLICATION INSTRUCTIONS

1. Release all hook and loop closures and lay on flat surface with brushed loop facing downward (sizing tag will be facing upward).
2. Place palmer crease portion of hand on wide mid section of support with fingers over extension pieces and thumb below the wide mid section.
3. Grasp ends of wide mid section, wrap over across back of hand and secure hook and loop closure.
4. Starting with web space between index finger and middle finger, grasp extension piece, pass through web space, pull diagonally toward radial edge of hand and secure to back of hand.
5. Continue in the same process, wrap each extension piece through the next web space, pull diagonally and secure. Note: the last extension piece should wrap around side of the little finger, pulled diagonally and secured.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.