



CORFLEX.
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HIP MISC

HIP ABDUCTION PILLOW

- > Features concave shape to ensure comfortable positioning and anatomical fit
- > Indicated for post-operative hip positioning
- > Manufactured from sturdy medium-density foam with soft foam straps and contact closure



PRODUCT NUMBER

S	M	L
89-1411	89-1412	89-1413

SIZING GUIDE

S	M	L
Width/Length 12" x 17"	14" x 22"	16" x 25"

APPLICATION INSTRUCTIONS

1. Release all hook and loop closures.
2. Grasp pillow by curved-in sides, position pillow between legs with curved-in sides against inside of each leg, making sure tapered end is closest to torso.
3. Secure hook and loop closures to desired tightness. For a customized fit, the plastic plates are bendable and may be gently squeezed for a smaller arm or flattened for a larger arm.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.