



CORFLEX

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



HIP MISC

NEXT STEP SUSPENSION BELT

- > Designed to provide trans-femoral amputee optimal comfort, suspension and rotational control
- > Ideal form of primary suspension, ancillary rotational control for lanyard and liner wearers
- > Manufactured from latex-free soft-grip neoprene against the skin with durable nylon outer layer and heavy-duty 2" nylon straps with contact closure



PRODUCT NUMBER

24"	26"	28"	30"	32"	34"
89-1450	89-1451	89-1452	89-1453	89-1454	89-1455

36"	38"
89-1456	89-1457

SIZING GUIDE

24"	26"	28"	30"	32"	34"
2" x 24"	2" x 26"	2" x 28"	2" x 30"	2" x 32"	2" x 34"

36"	38"
2" x 36"	2" x 38"

Measurement taken from anteriorly-mounted chafe across opposite hip to posterior chafe, subtract 6" from measurement

APPLICATION INSTRUCTIONS

1. After attaching two 1 1/2" chafes to your desired locations you are ready to attach the Next Step Belt.
2. Open hook and loop on one strap and feed through posterior chafe.
3. Open hook and loop on other strap and attach through anterior chafe.
4. Straps can now be adjusted to individual for optimal rotational control.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Close all contact closures and hand wash in warm water with mild soap. Product should be air-dried, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.