



CORFLEX.
.....

**CORFLEX IS A GLOBALLY BASED
MANUFACTURER AND DISTRIBUTOR OF QUALITY
ORTHOPEDIC AND REHABILITATION PRODUCTS**



TENNIS ELBOW SUPPORTS

BANDIT ELBOW STRAP

- > Unique design provides pressure over affected tendons while preserving complete range of motion of the elbow
- > Indicated for treatment of medial or lateral epicondylitis and carpal tunnel syndrome
- > Constructed of two-piece rigid padded plastic with contact closure strap



PRODUCT NUMBER

UNIV
99-5600

SIZING GUIDE

UNIV
6"-15"

Measurement taken at widest part of forearm

APPLICATION INSTRUCTIONS

1. Place BandIT on the forearm.
2. Tighten the strap, pulling it from the underside of your arm towards you.

Note: Tighten the strap no more than is necessary to keep BandIT in place during the activity that aggravates your condition (i.e., make it tighter for tennis, looser for typing). When worn, the strap between the two plates should be equal in length on each side. The plate with the logo sits on top of the arm; the other, under the arm. To achieve this, pull the bottom plate (the plate without BandIT's logo) along the strap. The strap is not glued to the bottom plate.

For a more customized fit, the plastic plates are bendable and may be gently squeezed for a smaller arm or flattened for a larger arm.

CARE & CLEANING

Corflex manufactures a quality line of products and to help your care for the product properly we provided the following instructions for care & cleaning:

Wipe plastic shell with damp cloth, you may use disinfectant or mild soap, and wipe dry.

For products with a removable liner, you may hand wash liner in warm water with mild soap, air dry, do not use high temperatures to wash or dry.

HOW TO PLACE AN ORDER

The Corflex Customer Service Team is ready to assist you in placing Your order. You may place your orders via a variety of methods:



TELEPHONE

1-800-426-7353, Ext1



FAX

1-603-623-4111



E-MAIL

Service@corflex.com



MAIL

CORFLEX, Inc.
Attn: Customer Service
669 East Industrial Park Dr
Manchester, NH 03109 USA

669 East Industrial Park Drive - Manchester, NH 03109
Phone (603) 623-3344 - Fax (603) 623-4111 - www.corflex.com

© Corflex | The advice and information contained in this document is not intended as a substitute for medical counseling. If you have any questions about treatment for a specific condition, please check with a health care professional.